Dear customer,  
Thank you for placing your trust in us. This footwear is a Class I Medical Device (MD) that complies with the provisions of Regulation (EU) 2017/745 on medical devices. The CE marking guarantees the product's quality and safety.

### Description

Celia Ruiz Calzados has designed and manufactured this footwear with a last having a special width. We use high-quality technical fabrics for both the inside and outside of the shoe, offering hypoallergenic and bacteriostatic properties. The sole is made from a high-density rubber having excellent abrasion resistance and non-slip properties during normal conditions, to ensure a good grip when walking. The removable insole is made from a viscoelastic base material.

#### 1. Application - Intended use

**⚠ If you have any questions, please contact a healthcare professional.** CELIA RUIZ® footwear is designed for individuals with chronic systemic diseases affecting their feet:

- Diabetes mellitus: ulcers, wounds, edema, blisters.
- Circulatory diseases: ulcers, wounds, edema.
- Neurological diseases that affect walking (Alzheimer's disease, paralysis of the leg nerves).
- Rheumatic diseases: Arthritis or Arthrosis.
- Traumas: Toe deformities, osteoporosis, metatarsalgia, plantar fasciitis, tendonitis, tarsal coalitions.
- Skin diseases: viral, bacterial and fungal infections.
- Postoperative period: Heavy bandages, sprains, casts, plantar orthoses.

#### 2. Place and conditions of use:

The outdoor footwear should only be worn outside and the indoor footwear is only for inside use. Outdoor models are designed for use on the street. Dry the footwear if worn in the rain (according to the instructions from section 8. *Cleaning and care*).

**⚠ Note that there is a risk of slipping on wet or slippery surfaces, or if there is snow or ice on the ground. Always use extreme caution in these situations.**

#### 3. Precautions

**⚠** Read these instructions carefully prior to using the product. Before use, make sure that the product coincides with the picture on the packaging and that it is free from damage.

Do not use the product if there are any deviations in its properties or characteristics and immediately report any alterations to the manufacturer.

#### 4. Warnings

**⚠** This CELIA RUIZ® footwear should not be used by individuals who are hypersensitive to any of the product's components. **This product is contraindicated for individuals with latex allergies; the sole is made of natural latex or may contain traces of natural latex.**

In the case of a serious incident (death or severely deteriorated health), notify the manufacturer and the relevant federal authority.

#### 5. Materials:

- Celia Ruiz footwear is made from a variety of materials:
- External fabric: textiles vary depending on the model.
  - Inner fabric or lining: 100% Coolmax/carbon polyester.
  - Removable insole: 100% Coolmax/carbon polyester on a synthetic latex base.
  - Sole: Natural rubber (latex).


#### 6. Correct product use

Prior to the initial use:

- Remove any filling material (paper, etc.) and adjust the footwear.
- Adjust the footwear (strap length) if necessary.

To use the footwear, follow these instructions:

- 1 Sit on a sturdy chair
- 2 Open the adjustable straps
- 3 Place your foot inside the shoe
- 4 Close the strap and adjust it to your foot, making sure that it is neither too tight nor too loose.

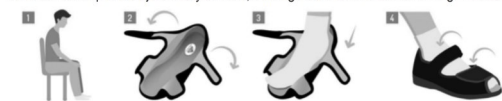Do not overtighten the velcro straps to avoid creating areas of pressure or reduced blood flow. If the straps are too loose, the footwear will slide around, possibly leading to falls or pressure sores.

This footwear is a medical device intended for the exclusive use by a single patient. The footwear is not authorised for use by anyone else.

The duration and frequency of use should be determined by a healthcare professional.

#### 7. Safety instructions.

**⚠** Before each use, inspect the inside of the footwear for any foreign objects.  
Do not use Celia Ruiz footwear without insoles or without insoles prescribed by a specialist.  
Always wear socks with the Celia Ruiz footwear.

At first, only wear new footwear for short intervals, checking for any areas of pressure after approximately 20 minutes of use. A healthcare specialist will determine the interval and duration of daily use.  
If you notice areas of pressure, pain or other problems, immediately contact the healthcare specialist in charge of your treatment who prescribed the footwear.  
Check your feet daily for any redness or areas of pressure.

#### 8. Cleaning and care

##### Cleaning and disinfection.

Use a neutral soap for delicates to clean the footwear's fabric surfaces. Hand wash using warm water and rinse well. A soapy film may appear the first few times that you wash the removable insole. This film will not damage the footwear's properties, but it should not come into contact with mucous membranes or open wounds.

##### Drying.

Dry away from heat sources (never use a hair dryer or radiator to dry).

Do not use heaters to dry the footwear or expose to direct sunlight.

Allow sufficient time for the footwear to dry naturally. Several days may be needed for drying, depending on the amount of moisture absorbed.

#### 9. Storage

Always store in a cool and dry location. Store the product safely in its original packaging, away from bright light.

#### 10. General indications / Maintenance.

Regularly check the footwear's capacity to function as intended, as well as the state of wear and deterioration of the fastening, sole and fabrics.

If detecting any loss of functionality, wear or damage, stop using the footwear and contact your healthcare specialist. Schedule regular appointments to check the footwear with your healthcare specialist.

We also recommend that you receive regular check-ups on your feet, in accordance with your diagnosis.

#### 11. Waste management

At the end of the product's useful life, dispose of the product and its packaging according to applicable waste management regulations.

#### 12. Warranty

This medical device has a legal warranty for the correct use of the footwear.

#### 13. Incident reporting

Patients should report any serious incidents resulting from the use of this medical device to Celia Ruiz Calzados, S.L. and/or the relevant healthcare authority in the country where they reside.

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